



# OTIDRY-MD<sup>®</sup>

## ear spray

### **INSTRUCTIONS FOR USE**

Please, read these instructions for use carefully before you start using this product. Keep them, you may need to read them again. Do not share this product with any other person. If you require additional information or have any further questions, ask your doctor or pharmacist.

### **What OTIDRY-MD<sup>®</sup> ear spray is and what it is used for:**

OTIDRY-MD<sup>®</sup> dries water from the external ear canal and relieves water-clogged ears due to bathing, showering, swimming, etc. thereby relieving discomfort, the sensation of fullness or hearing impairment.

OTIDRY-MD<sup>®</sup> creates the right conditions to prevent the harmful effects of excessive moisture in the ear, such as bacterial proliferation, which can lead to otitis externa (also known as swimmer's ear).

Thanks to the combination of the ingredients, OTIDRY-MD<sup>®</sup> has a triple action in the external ear canal:

- Evaporates the excess of water
- Preserves the physiological pH
- Soothes the skin

OTIDRY-MD<sup>®</sup> comes in a 30ml bottle with an ear dispensing device.

### **Who can use OTIDRY-MD<sup>®</sup> ear spray:**

OTIDRY-MD<sup>®</sup> is suitable for use in adults and children from 9 years old.

### **Do not use OTIDRY-MD<sup>®</sup> ear spray if:**

- you are allergic to any of its ingredients;
- you suffer from any form of otitis (for example: pain, infection, discharge);
- you have, or suspect that you may have, a perforated eardrum;
- you have tympanostomy (ventilation) tubes;
- you have had a recent ear surgery unless specifically cleared by the doctor;
- you have any open wounds or damaged skin in the external ear canal;
- you are pregnant or nursing.

### **Using other products or medicines with OTIDRY-MD<sup>®</sup> ear spray:**

OTIDRY-MD<sup>®</sup> will not interact with any other product or medicine that you may be taking.

However, do not administer it together with any other ear products. In case of doubt ask your doctor or pharmacist.

### **How to use OTIDRY-MD<sup>®</sup> ear spray:**

OTIDRY-MD<sup>®</sup> has to be used just after exposure or contact with water. It is applied using an ear dispensing device specially designed for use in the external ear canal as shown in the drawings below.


If you use hearing aids, you must be remove them before using OTIDRY-MD<sup>®</sup>

Dry your external ear canal with a soft towel or cloth. Don't insert the cloth into the canal.

Before using the product, assemble the ear dispensing device with the bottle by exerting slight pressure, and spray repeatedly until the liquid comes out.




**1.** After exposure or contact with water, tilt the head so the ear faces upward and gently pull the ear backwards.

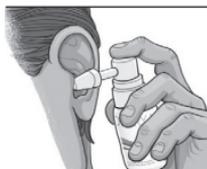

**2.** Carefully place the ear dispensing device in the external ear canal and press once. Wait 30 seconds.



**3.** Turn the head in the opposite direction to allow the liquid to drain and dry. Any remaining liquid will evaporate in a few minutes.

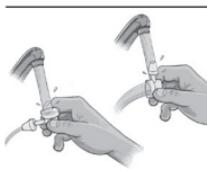

**4.** After each use, remove the ear dispensing device and rinse it with plenty of water at both ends to keep it clean and in optimal use conditions. Dry it and keep it for future applications.

Keep the bottle and ear dispensing device disassembled inside the carton for proper conservation.

**Important:** Before each use, spray repeatedly out of the external ear canal, until the liquid comes out. Then follow the steps indicated in drawings 1 to 4.

**How often to use OTIDRY-MD® ear spray:**

OTIDRY-MD® can be used whenever necessary up to a maximum of twice a day.

**Possible side effects:**

Just because a side effect is stated here, it does not mean that all people using this product will experience that or any other side effect.

It has been reported that ear solutions may cause discomfort, pain, transient hearing loss (due to a liquid presence in the ear canal), dizziness, skin irritation.

In case of excessive pain or discomfort or loss of hearing are noticed, as well as any other possible side effects not mentioned before, you should stop using the product and seek medical attention.

**Warnings and precautions:**

For external use only, exclusively to be used in the ear.

In case of contact with eyes, rinse immediately with plenty of water.

Do not swallow.

In case of poisoning, contact the National Toxicology Service.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State

**How to store OTIDRY-MD® ear spray:**

Keep out of reach of children.

Keep the product away from heat sources, sparks, open flame or hot surfaces.

Store below 30 °C.

Store in an upright position.

Do not refill the container.

Do not freeze.

Do not use after the expiry date printed on the carton and bottle, even if the product has not been used.

Use within 12 months after opening.

**Composition:**

Ethanol 70%, boric acid 3%, glycerol 3%.

**Date of last revision of the instruction for use:** October 2021



Under License from Mitelos Bioscience S.L.



**Manufacturer:**  
Mitelos Bioscience S.L.  
Parc Tecnològic del Vallès. Ronda de Can Fatjó 7-B  
08290 Cerdanyola del Vallès, Barcelona, Spain

